
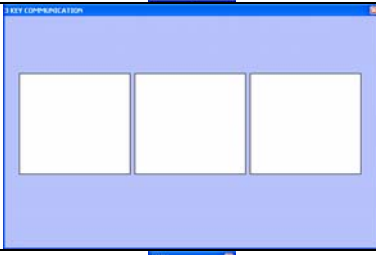
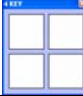





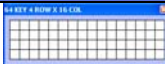



# Default Discover:Screen Templates and Setups

## Discover:Screen Templates (Sorted Alphabetically)

Template Name	Picture	Description
2 Key.tmo		Blank Discover:Screen Template
3 Key Communication.tmo		Blank Discover:Screen Template
4 Key.tmo		Blank Discover:Screen Template
8 Key 2 Row X 4 Col.tmo		Blank Discover:Screen Template
8 Key Vert 4 Row X 2 Col.tmo		Blank Discover:Screen Template
15 Key 3 Row X 5 Col.tmo		Blank Discover:Screen Template
16 Key 2 Row X 8 Col.tmo		Blank Discover:Screen Template
32 Key 4 Row X 8 Col.tmo		Blank Discover:Screen Template
64 Key 4 Row X 16 Col.tmo		Blank Discover:Screen Template
76 Key.tmo		Blank Discover:Screen Template

### Default Discover:Screen Templates and Setups

Madentec Limited, 4664 – 99th Street, Edmonton, Alberta, Canada T6E 5H5

(877) 623-3682 • [www.madentec.com](http://www.madentec.com)

Enabling Extraordinary People to do Ordinary Things!



ABC.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>
ABCPunct.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>
Frequency.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>
FrequencyPunct.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>
Qwerty.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>
QwertyPunct.tmo		<p>This is a mostly completed template, providing the ability to quickly make a variety of setups. All of the routine keys of the keyboard are completed to save a great deal of editing time. There are several blank keys to customize to whatever program/purpose. An unlimited number of setups may be created based on this template.</p>

**Default Discover:Screen Templates and Setups**









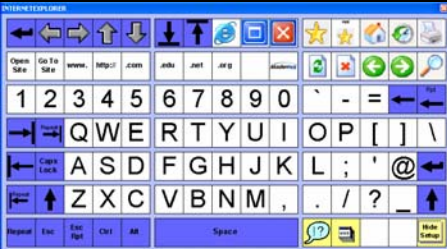
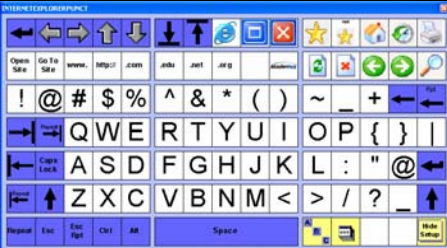
Madentec Limited, 4664 – 99th Street, Edmonton, Alberta, Canada T6E 5H5

(877) 623-3682 • [www.madentec.com](http://www.madentec.com)

Enabling Extraordinary People to do Ordinary Things!



# Discover:Screen Setups (Sorted Alphabetically)

Setup Name	Picture	Description
Advanced ABC-123.suo		ABC layout for word processing, E-mail, data entry, text applications. Branches to Application Launcher.
Advanced ETA-123.suo		ETA (frequency of use) layout for word processing, E-mail, data entry, text applications. Branches to Application Launcher.
Advanced Qwerty.suo		QWERTY layout for word processing, E-mail, data entry, text applications. Branches to Communication-Start Here, and Application Launcher.
Application Launcher.suo		Provides direct access to launching common programs built into the operating system including Internet Explorer and Outlook Express. Many blank areas left available to add additional programs to launch (first group) and branch to if needed (third group). Branches to Application Launcher Vol 1 and Application Launcher Vol 2.
Communication-Start Here.suo		Communicating while working on the computer. Branches to Let's Talk, NeedsWants, and Advanced QWERTY.
Easy abc-123.suo		Basic word processing and numbers programs; contains lower-case letters on the setup. Branches to Application Launcher.
Easy Numbers.suo		Basic number programs. Branches to Application Launcher.
Hide Setup.suo		This setup is used in conjunction with most of the rest of these setups. The other setups have a key designated with Hide Setup which in turns branches to this small setup. This is a single key setup with a key that branches back to the last setup used.
InternetExplorer.suo		Internet Explorer access. Branches to InternetExplorerPunct, and Application Launcher.
InternetExplorerPunct.suo		Internet Explorer access. Branches to InternetExplorer, and Application Launcher.




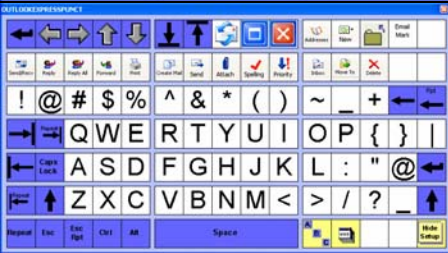



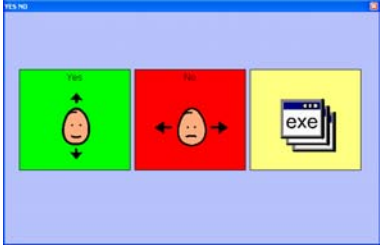
## Default Discover:Screen Templates and Setups

Madentec Limited, 4664 – 99th Street, Edmonton, Alberta, Canada T6E 5H5

(877) 623-3682 • [www.madentec.com](http://www.madentec.com)

Enabling Extraordinary People to do Ordinary Things!



Let's Talk.suo		Communicating quick messages while working on the computer. Branches to Communication-Start Here.
NeedsWants.suo		Communicating simple messages of needs and wants while working on the computer. Branches to Communication-Start Here.
OutlookExpress.suo		Outlook Express access. Branches to OutlookExpressPunct, and Application Launcher.
OutlookExpressPunct.suo		Outlook Express access. Branches to OutlookExpress, and Application Launcher.
Small Advanced Qwerty.suo		Standard, but small, QWERTY layout. Branches to Communication-Start Here, and Application Launcher.
Word Bank-Pets1.suo		Word bank setup that speaks words and helps in the creation of sentences while in word processing document. Branches to Word Bank-Pets2.
Word Bank-Pets2.suo		Word bank setup that speaks words and helps in the creation of sentences while in word processing document. Branches to Word Bank-Pets1.
Yes No.suo		A simple setup used in evaluating if a student can answer yes/no questions. Easily adapted for choice making activities, etc. Branches to Application Launcher.

**Default Discover:Screen Templates and Setups**

Madentec Limited, 4664 – 99th Street, Edmonton, Alberta, Canada T6E 5H5

(877) 623-3682 • [www.madentec.com](http://www.madentec.com)

Enabling Extraordinary People to do Ordinary Things!

