

Discover Introduction

Basic Conventions and Terminology

1. Discover on screen displays or Discover:Board/Key Largo overlays will be referred to as setups.
2. The individual items displayed on the setups will be referred to as keys.
3. A scanning setup may contain several groups and several subgroups of keys.
4. A group is a set of keys – either a row or column – in a scanning setup that is highlighted after activating Discover.
5. A subgroup is a set of keys in a scanning setup that are scanned after selecting a group.
6. Setups with groups and subgroups of keys may be set to scan in either the row/column or column/row method.
7. Each piece of software may have a setup “attached” – meaning that the attached setup is the one that will be active by default each time the application is launched. However, additional setups may be used with each application by either opening them through the Discover Menu or through the use of branching keys.
8. Branching is when a key on a setup is designated to open up a different setup to be used with the current application. It is critical that setups branch to and branch back in a “two way” fashion so that the user will be able to return to the original setup without having to exit the current application.
9. A script is the automation of more than one computer task. Scripts may be programmed onto keys in a setup.
10. Exact “clickable” areas of the screen will be designated through the use of Smart Markers

A partially created scanning setup for *WordPad* has been created and named *WordPad Partial.sus*. It is named as such, because the setup is partially done. These tutorials will give you the instructions needed to complete this setup. As a reference, another completed setup has been created and named *WordPad Final.sus*.

Throughout these tutorials text entry and/or commands will be required. Text items will be presented within double quotation marks, with the “**text**” bolded. Do not type the quotation marks or make any of the text bold. When instructed to *click*, please use the mouse. When instructed to *activate*, please use Discover. Keyboard shortcuts, where applicable, will be provided in **(parenthesis)**.

A Discover setup may only be edited only if it is the current active setup. In other words, only the open setup with the current application running in the forefront of the computer may be edited. An application may have a particular setup already attached to it (by default) or an application may have no setup attached. Applications without a setup attached may still be used with Discover; however, a setup will need to be opened each time through the use of the Discover Menu.

These tutorials assume that Discover:Kenx hardware and software are being used. The steps may differ slightly for another Discover product (i.e., Discover:Switch).

Discover Introduction

Madentec Limited, 4664 – 99th Street, Edmonton, Alberta, Canada T6E 5H5
(877) 623-3682 • www.madentec.com

Enabling Extraordinary People to do Ordinary Things!

