

Discover and the Mouse

Using Discover for Complete Mouse Emulation

The Application Launcher

The **Application Launcher.sus** is a scanning setup to be used to launch programs through Discover access once the computer is powered on and is at the desktop. This setup is attached to EXPLORER.EXE and USER.EXE.

The **Application Launcher.sus** setup includes 4 groups of keys with two subgroups in each group. The first group includes the keys designed to launch individual pieces of software and provides branch keys to additional Application Launchers that are included with Discover:Setups Volume I and II CD's. The second group includes keys for mouse single click, mouse pointer movement, Enter, Scan Buttons, and Shut Down. The third group includes the keys for mouse click functions (**Single Click**, **Double Click**, **Drag On**, **Drag Off**, and **Right Click**), and mouse clicking and dragging with movement functions (the green keys). The fourth group includes keys for the purpose of branching to the default setup for the corresponding software.

Mouse Control

Mouse control keys are provided on several setups (scanning or alternative keyboard) because access to the mouse may be required at different times with different applications. For this example, the **Application Launcher.sus** scanning setup is pictured below.

To Move the Mouse Pointer

1. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the mouse pointer navigation keys. (Group 2, Subgroup 1)
2. Activate **Discover** when the desired mouse pointer navigation key (**Mouse Left**, **Mouse Right**, **Mouse Up**, or **Mouse Down**) is highlighted. (The mouse pointer will continuously move in the selected direction.)
3. Activate **Discover** when the mouse pointer has reached the desired destination.



To Click or Double Click the Mouse Button

1. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the **Click** and the **Double Click** keys. (Group 3, Subgroup 1)
2. Activate **Discover** when the **Click** or the **Double Click** key is highlighted.



To Execute the Click and Drag Functions of the Mouse

1. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the **Begin Drag** and the **End Drag** keys. (Group 3, Subgroup 1)
2. Activate **Discover** when the **Begin Drag** key is highlighted.
3. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the mouse pointer navigation keys. (Group 2, Subgroup 1)
4. Activate **Discover** when the desired mouse pointer navigation key (**Mouse Left, Mouse Right, Mouse Up, or Mouse Down**) is highlighted. (The mouse pointer will continuously move in the selected direction with the mouse button down in drag mode.)
5. Activate **Discover** when the mouse pointer has reached the desired destination (e.g., the desired text is highlighted, or the icon is moved to the desired location).
6. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the **Begin Drag** and the **End Drag** keys. (Group 3, Subgroup 1)
7. Activate **Discover** when the **End Drag** key is highlighted. (This will complete the click and drag process and set the mouse back to its standard mode).



To Move the Mouse Pointer with Drag Mode

1. Activate **Discover** and navigate the scanning to the **group** or **subgroup** that contains the mouse pointer drag keys. (Group 3, Subgroup 2)
2. Activate **Discover** when the desired mouse pointer drag key (**Drag Left, Drag Right, Drag Up, or Drag Down**) is highlighted. (The mouse pointer will continuously move in the selected direction with the mouse button down in drag mode.)
3. Activate **Discover** when the mouse pointer has reached the desired destination (e.g., the desired text is highlighted, or the icon is moved to the desired location).

